Comfiwave®

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www.comfiwave.info

British Journal of
Community Nursing

PRODUCT FOCUS

Circumference (cm)	A	C	D	F	G	Limb volume A	Unaffected	% difference
Baseline	17.5	17.5	25	29	31.5	2329	1954	+19%
After 2 weeks	17	15.8	24	27.5	31	2139	1953	+8%

through her teenage years with bilateral lower leg oedema and recurrent episodes of cellulitis.

On initial assessment, there was evidence of bilateral lymphoedema below the knee, with both feet showing positive Stemmer's sign, mild hyperkeratosis and post-cellulitis skin discolouration on the left gaiter area. The left leg was 30% bigger than the right, with an ankle circumference of 31 cm and calf of 45 cm. There were distinct skin folds distorting the shape around her left ankle, as well as foot oedema, which was restricting footwear (Figure 4). The limb size also restricted clothing, and the patient was only able to wear very loose trousers or long skirts.

Initial advice of skin care, exercise, elevation and weight management were discussed, and 2 weeks of daily treatment were booked. Miss H was signed off work by her GP to enable her to attend the sessions. Daily treatment commenced in October 2018 and involved alternating days of MLD or intermittent compression using Haddendum LymphFlow Advance, followed by below-knee multi-layer compression bandaging. Due to the extensive fibrotic

Figure 5. Miss H wearing Comfiwave on the left leg

Comfiwave under Easywrap for 2 weeks to compare this with the previous Minkiderm/Easywrap combination (Figure 5).
The feedback after 2 weeks was that it was a much more comfortable garment combination to wear, and the combined Minkiderm/Easywrap...

Figure 6. Miss H's legs after using Comfiwave for 2 weeks

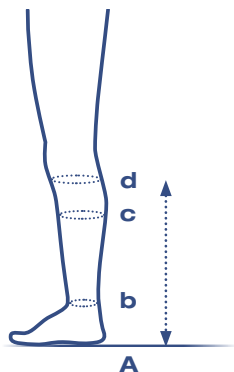
within
wave
tients'
to the
y were
improve
ment of
that this
gent or as
24 hours
efficacy in
decreasing
oedema and
gent, family

ent: a prospective
3-541. <https://doi.org/10.1016/j.jbv.2017.04.001>
ed sleep with using
Coman Nurs. 2017;
staker J. Examination
in tissue oxygenation
429-435. <https://doi.org/10.1016/j.jbv.2017.04.001>
© JovVita PwPb in a
the British Lymphology

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Haddenham Healthcare Ltd